

Ahlstrom-Munksjö GenSwab

Collect and preserve oral/buccal samples for decades

GenSwab is a product designed for easy collection and long-term preservation of buccal sample to achieve high recovery of DNA and high quality genetic profiles.

GenSwab results from the collaboration between Ahlstrom-Munksjö and Gentuери, two companies with several decades of experience in the design of products and custom kits. Our products allow preservation of bio samples at ambient temperature, safeguarding their quality over time for downstream processing.

We are committed to providing our customers with innovative and cost-efficient products, for an effective and simplified sample collection process.


Get the most out of your samples by ensuring they are well collected, identified, preserved and stored.



Key features and benefits

- Easy collection and effective transfer of sample from swab onto specimen card
- Long-term preservation of DNA on GenSaver™ 2.0 Color paper at ambient temperature
- Compliance with the ISO18385:2016
- Environmentally friendly paper-based design
- Automated design compatible with automated DBS equipment
- Complete protection of the sample during the transport
- Improved sample traceability with multiple barcodes

Product Characteristics

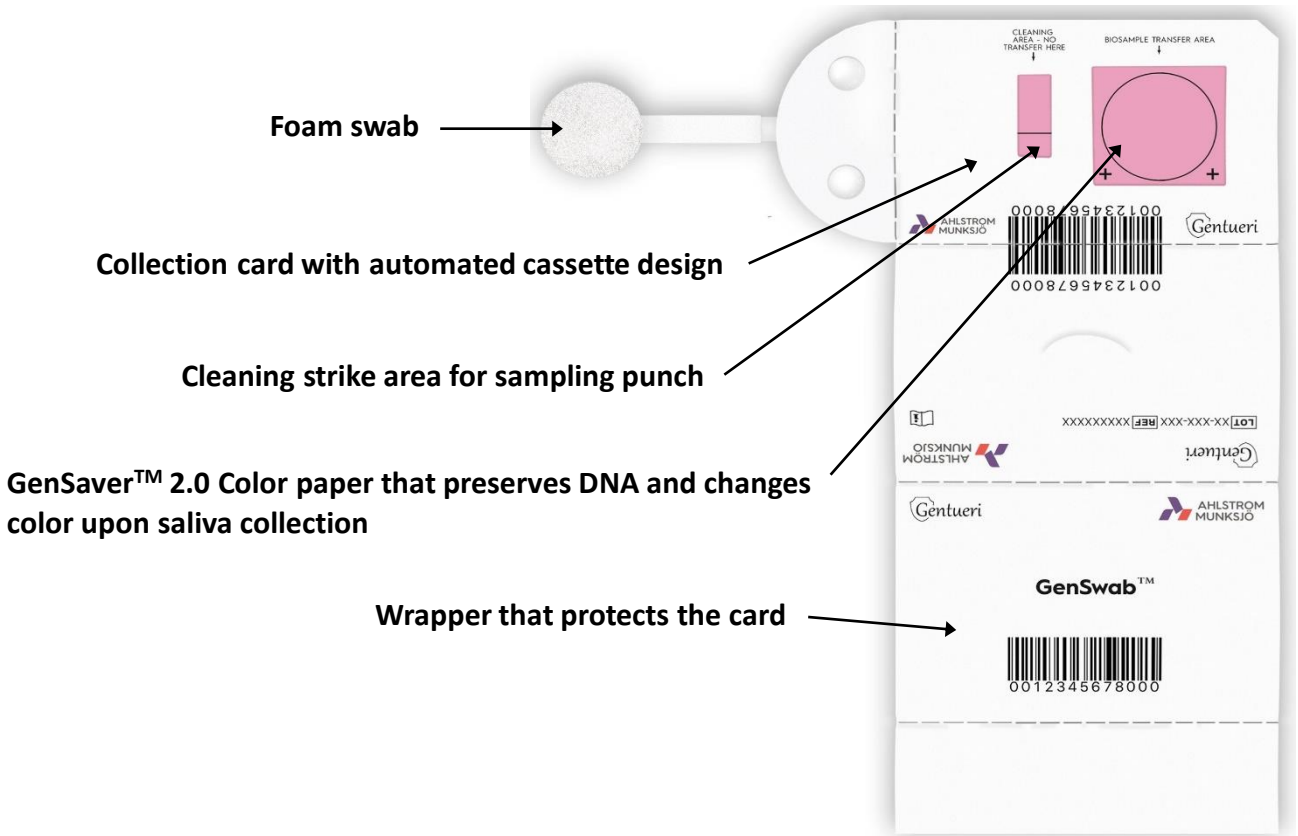

Collection process

1. Take GenSwab from sterile pouch
2. Place thumb as indicated on the hard card and swab the inside of your cheeks for 20 seconds on each side
3. Unwrap the card and flip the swab over the preservation paper
4. Fold wrapper back over the card and squeeze the foam onto the card to transfer the sample
5. Unwrap the swab so it can be removed at the perforation, wrap the card and place it in the GenPouch™ or other appropriate envelop

CONTACT AHLSTROM-MUNKSJÖ SALES

EMEA

+49 37347 830

laboratory@ahlstrom-munksjo.com

North America

Toll-free

+1 800 326 1888

laboratory@ahlstrom-munksjo.com

Customer Service

+1 717 486 3438

laboratory@ahlstrom-munksjo.com

LEARN MORE: WWW.AHLSTROM-MUNKSJÖ.COM

DISCLAIMER: The information supplied in this document is for guidance only and should not be construed as a warranty. All implied warranties are expressly disclaimed, including without limitation any warranty of merchantability of fitness for use. All users of the material are responsible for ensuring that it is suitable for their needs, environment and end use. All data is subject to change as Ahlstrom-Munksjö deems appropriate.